

→ MECTRON PIEZOSURGERY®
RHINOPLASTY PROCEDURES
REDUCED TO THE MAXIMUM



→ FOCUS ON SIMPLICITY



- Simple protocol
- Use Piezosurgery only for steps where it really makes the difference

→ RHINOPLASTY WITH PIEZOSURGERY® INSERTS.
USE THE ORIGINAL! KEEP IT SIMPLE!



HUMP TAKEDOWN (RASPING)

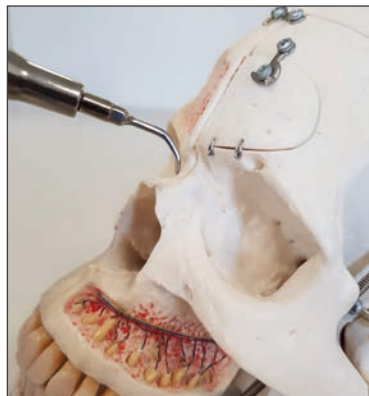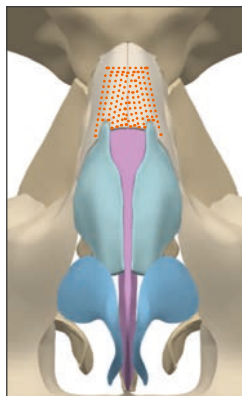

- Progressive removal of bone;
- Controlled gentle rasping of the nasal dorsum;
- Non traumatic - reduce trauma to the bone and spares surrounding tissues.


OSTEOTOMY

A: medial oblique + low to low

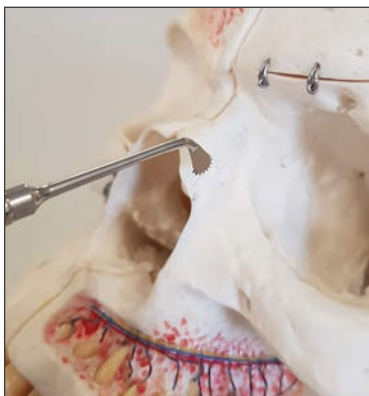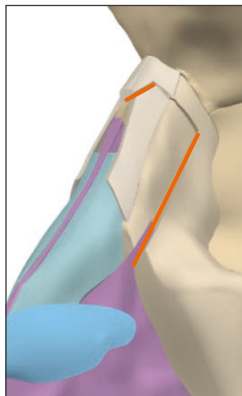

- Non traumatic - prevents nasal mucosa or periosteum lacerations, spares soft tissue and cartilage;
- Controlled action;
- Easy to handle;
- No radiating fracture lines.


B: transverse (optional step)

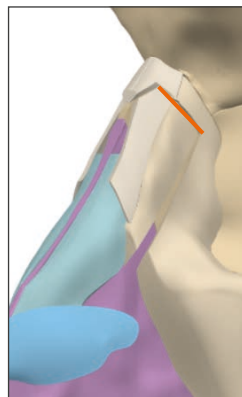


- R and L curvature for precise action in a limited space.

→ RHINOPLASTY WITH PIEZOSURGERY® INSERTS.
USE THE ORIGINAL! KEEP IT SIMPLE!

OSTEOPLASTY


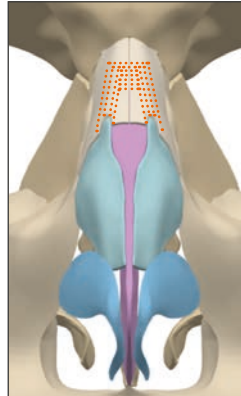


- Progressive removal of bone;
- Controlled gentle rasping of the nasal dorsum;
- Non traumatic - reduce trauma to the bone and spares surrounding tissues.

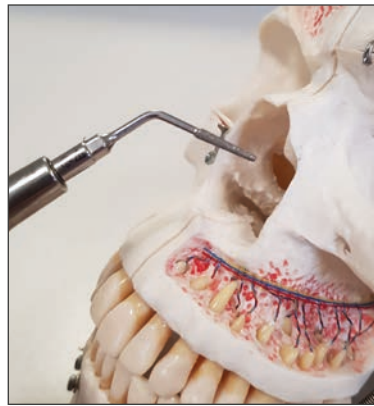

- Fine tip for small holes;
- Bent for insertion in limited space;
- Ideal for perforating nasal bones as well as ethmoid lamina splints.

SEPTOPLASTY

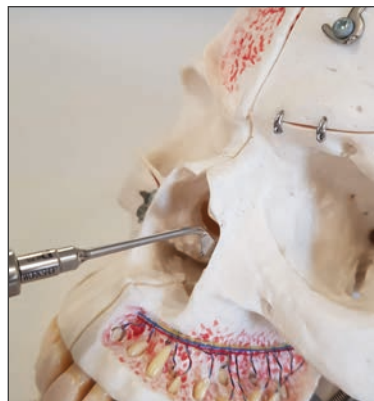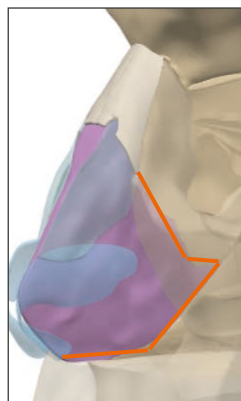

- Long curved insert will work in limited space;
- Makes septoplasty safe even for posterior bony deviations;
- Allows precise work on vomer and spine.

→ RHINOPLASTY WITH PIEZOSURGERY® INSERTS. USE THE ORIGINAL! KEEP IT SIMPLE!

→ INSERTS PIEZOSURGERY® - SINGLE USE



PIEZOSURGERY® flex → THE PERFECT START

→ INTRAOPERATIVE ADVANTAGES

- **Selective Cut** Maximum safety for surgeons and patients. Reduced risk of damaging soft tissues (dura, nerves and vessels).
- **Micrometric Cut** Maximum surgical precision and intra-operative tactile sensation. Minimal bone loss through the cutting width.
- **Cavitation Effect** Maximum intra-operative visibility. Blood-free surgical site.

→ POSTOPERATIVE BENEFITS

- **Healing** Better and faster bone healing.
- **Edema** Often reduced, due to full respect of soft tissues.



mectron s.p.a.,
via Loreto 15/A, 16042 Carasco (Ge), Italy,
tel +39 0185 35361, fax +39 0185 351374

→ www.mectron.com or piezosurgery@mectron.com

© Copyright mectron S.p.A., Carasco, Italy
All rights reserved. Texts, pictures and graphics of mectron brochures are protected by copyright and other protection laws. Without written approval of mectron S.p.A. the contents may not be copied, distributed, changed or made available to third parties for commercial purposes.